
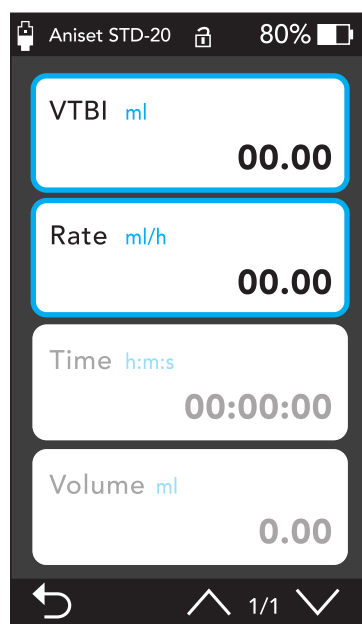


We have written this quick start guide for use with the mls/hr mode, in order to allow the user to deliver an infusion as quickly as possible.

**You must use electrical power for its first use and allow the battery to fully charge before using battery power.**  
**You can only use a pre set Aniset™ line: to use a different line that is not calibrated for use will lead to inaccurate fluid delivery.**

- 1 Plug the Anifusion® IV Pump into an electrical supply.
- 2 Press and hold the power button located at the bottom left hand corner of the pump.
- 3 On Main menu, select modes.

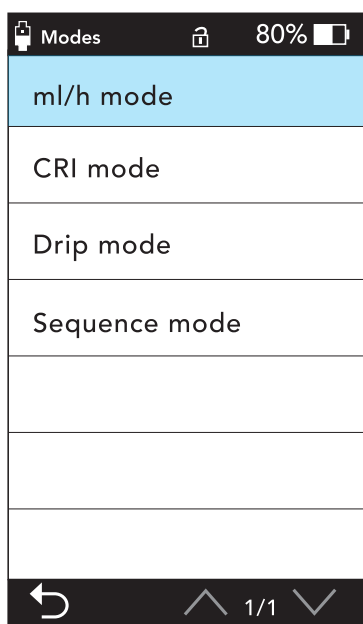

- 4 Select ml / hr mode

Select VTBI (volume to be infused) and using keypad, enter the amount of fluid required.

Press  when complete.

- 5 Select Rate

Using the keyboard enter the mls/ hr rate you wish to use



Press  when complete.

Time will adjust itself based on amount of fluid and infusion rate you have entered.

6 Select volume

The Anifusion® IV Pump will ask if you wish to reset total volume.

Press

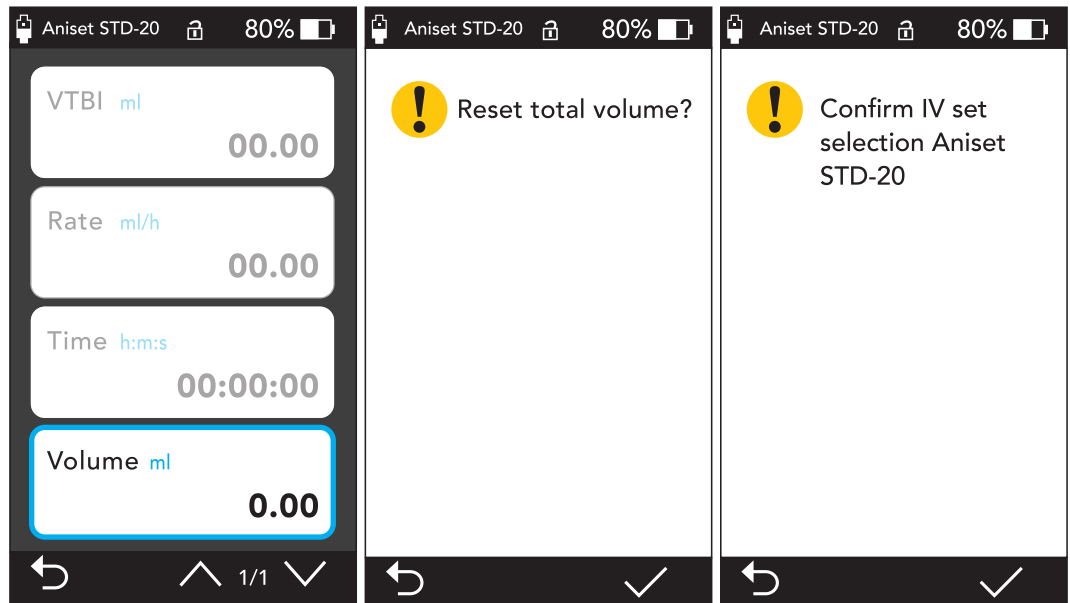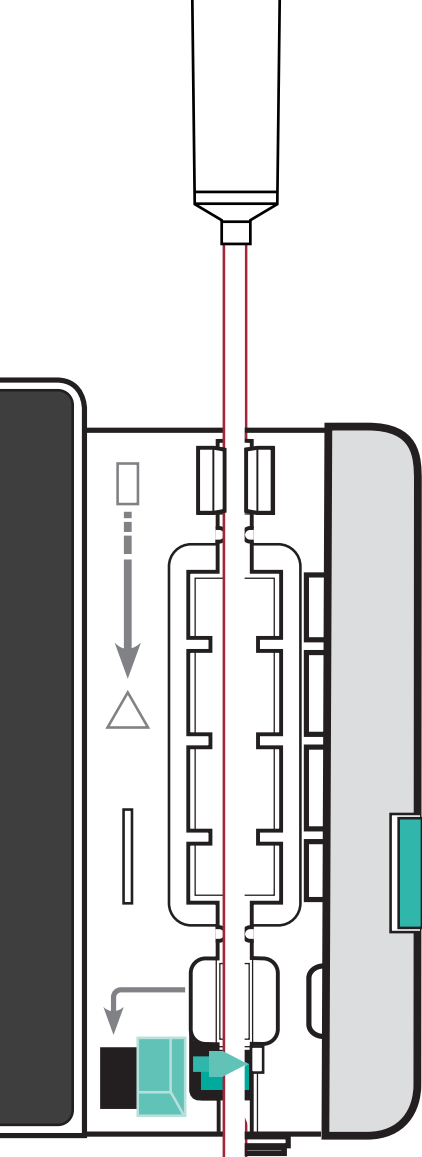

7 (Pump calibrated tubing) Open the door to the infusion pump and insert the primed line running from top to bottom.

Close the door ensuring it clicks shut.

8 The Anifusion® IV Pump will ask you to select which line you would like to use and will show you your options.

Confirm your selection

9 Press the start / stop button located on bottom right hand side of the pump to start the infusion.



Start/Stop  
Button



**MILLPLEDGE  
VETERINARY**

millpledge.com

United Kingdom: 08000 11 22 88

– sales@millpledge.com

USA: (888) 715-9909

– nasales@millpledge.com

Canada: (888) 715-9909

– casales@millpledge.com

Europe: +32 (0) 51 70 30 21

– eusales@millpledge.com

Australia: local: 1800 950 585

international: +61 1800 950 585 – ausales@millpledge.com