



Instructions for Cleaning Flow Meter Tube & Ball

1 Turn the oxygen supply to the machine OFF.

2 Remove the flow meter window by gently pulling it away from the flow meter.



3 Using an allen wrench, remove the plug in the top of the flow meter.



4 Remove the glass tube from the flow meter.



5 Using needle nose pliers, pull the plastic inserts from each end of the tube being careful not to drop the ball.




6 Wash the ball with alcohol. Wash the tube with soap and water using a small brush. Rinse with alcohol, and allow to air dry completely before re-assembling.

7 When completely dry, place the ball back into the tube and insert white plastic piece into the tube.

8 Insert tube back into flow meter.

9 If necessary, insert the tube seat guide back into bottom of plug.

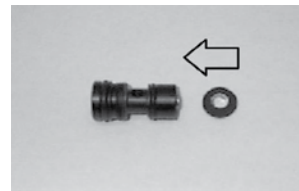

10 Insert plug into top of flow meter. Using an allen wrench, tighten plug carefully aligning with top of tube. Tighten just until tube is secure. **DO NOT OVER-TIGHTEN.**

11 Replace flow meter window.



Visit our Web Store for easy online ordering!

www.vetamac.com

1-800-334-1583



Find us



Follow us

130 Roth Court, Suite 100

PO Box 178

Rossville, IN 46065