

How To Spike Your Bag


Add Medication



Add Medication Before Solution Administration

- Hold the port straight
- Insert needle

Spike Bag

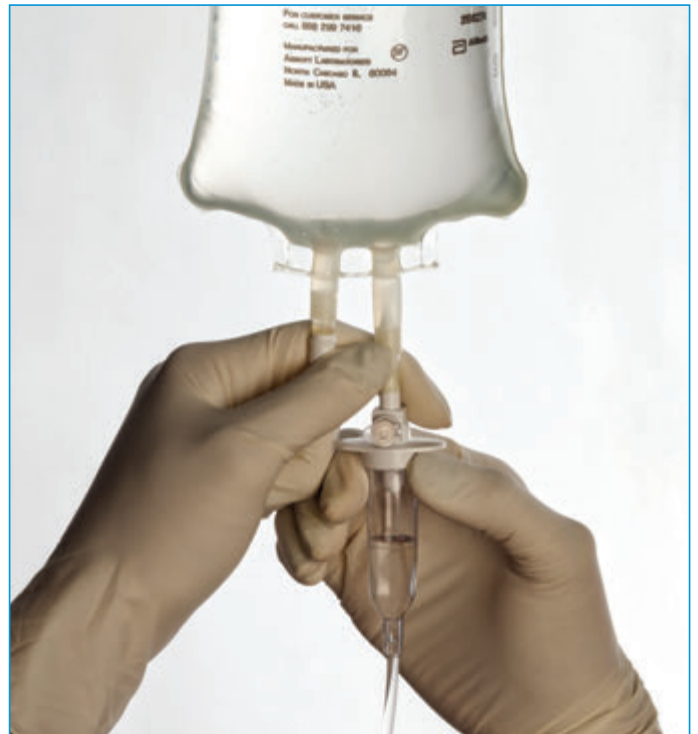

Attach and Remove Administration Set

- Hold the port straight
- Simultaneously PUSH and TWIST the spike

Detailed instructions on reverse.



How To Spike Your Bag

Step 1



Open Protective Packaging

- Locate the small slit at the eyelet (or top) of protective overwrap.
- Grasp protective overwrap with each hand on side of slit.
- Tear straight down to create a suitable opening.
- The plastic may exhibit some opacity or moisture, which can be attributed to the sterilization process. This is normal and does not affect the quality or safety of the solution, and will gradually diminish.
- Check for minute leaks by squeezing inner bag firmly. If leaks are found, discard solution, as sterility may be impaired.

Step 2



Add Medication Before Solution Administration

- Swab the medication port using aseptic technique.
- Use 19 to 22-gauge needle syringe to puncture medication port, ensuring needle fully penetrates the inner septum.
- Inject medication.
- Mix solution and medication thoroughly. For high density medication such as potassium chloride, squeeze ports while ports are upright and mix thoroughly.

Step 3



Attach and Remove Administration Set

- Use with a nonvented set or a vented set with the vent closed.
- Suspend container from eyelet support.
- Remove protector from the administration outlet port.
- Hold the administration outlet port straight while inserting the set spike to puncture the inset membrane.
- Continue the insertion by simultaneously PUSHING and TWISTING the spike in a single motion until spike is seated.
- To remove the administration set, simultaneously TWIST and PULL the set spike in a single motion.

WARNING: Do not use plastic containers in series connections. Such use could result in air embolism due to residual air being drawn from the primary container before administration of the fluid from the secondary container is completed.

NOTE: Product expires last day of the month indicated. Containers are latex free.